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<sup>ii</sup> BRAECKMAN (J.G.), AUTIER (P.), GARBAR (C.), PIPELEERS-MARICHAL (M.), SOVIANY (C.), NIR (R.), NIR (D.), MICHELSEN (D.), BLEIBERG (H.), EGEVAD (L.), EMBERTON (M.). 2007. Computer-aided ultrasonography (HistoScanning): a novel technology for locating and characterising prostate cancer. Published in British Journal of Urology International (BJUI). Vol. 101, Issue 3 (July), PP. 293-298, doi:10.1111/j.1464-410X.2007.07232.x.

<sup>iii</sup> BRAECKMAN (J.G.), AUTIER (P.), SOVIANY (C.), NIR (R.), NIR (D.), MICHELSEN (D.), TREURNICHT (K.), JARMULOWICZ (M.), BLEIBERG (H.), GOVINDARAJU (S.), EMBERTON (M.). 2008. The accuracy of transrectal ultrasonography supplemented with computer-aided ultrasonometry for detecting small prostate cancers. Published in British Journal of Urology International (BJUI). Vol. 102, Issue 11 (May), PP.1560-1565, doi:10.1111/j.1464-410X.2008.07878.x.

<sup>iv</sup> SIMMONS (L.A.M.), AUTIER (P.), ZÁT'URA (F.), BRAECKMAN (J.G.), PELTIER (A.), ROMÍCS (I.), STENZL (A.), TREURNICHT (K.), WALKER (T.), NIR (D.), MOORE (C.), EMBERTON (M.). 2011. Detection, localisation and characterisation of prostate cancer by Prostate HistoScanning™. e-Published 17/11/2011 in British Journal of Urology International (BJUI).

<sup>v</sup> SHIMKO (M.S.), KNOEDLER (J.J.), UMBREIT (E.C.), MYNDERSE (L.A.). 2010. Correlation of 3-dimensional ultrasound computer-aided interpretation with dynamic contrast enhanced pelvic MRI in the detection of post radical prostatectomy local recurrence of prostate cancer. Published and presented at the annual meeting of the American Urological Association (AUA). P.1002782, 29-03/05/2010, San Francisco (CA, USA).

- 
- <sup>vi</sup> EPPLEN (R.), van ESSEN (J.), van ERPS (T.), THÜER (D.), KNÜCHEL-CLARKE (R.), HEIDENREICH (A.). 2011. Detection of prostate cancer by HistoScanning. Published and presented at the annual meeting of the Société Internationale d'Urologie (SIU). Session "Podium Session 2: Prostate cancer, detection and screening", P.1432.00, POD-02.01, 17/10/2011, Berlin (D).
- <sup>vii</sup> LABANARIS (A.P.), ECK (A.), ADDALI (M.), AFRAM (S.), WITT (J.), ZUGOR (V.). 2011. The value of computer-aided ultrasonography in the detection and evaluation of prostate cancer. Published and presented at the annual meeting of the Société Internationale d'Urologie (SIU). Session: "Moderated poster session 3: Prostate cancer, detection and staging", MP-03.12, 17/10/2011, Berlin (D).
- <sup>viii</sup> SIMMONS (L.A.M.), LEMINSKI (A.), RASHID (T.), HAZELL (S.), MONZON (L.), WINKLER (M.). 2011. Blinded assessment of Prostate HistoScanning™ accuracy compared to elective radical prostatectomy step sectioned histopathology. Published at the International Symposium on Focal Therapy and Imaging in Prostate & Kidney Cancer (FTI-PKc). Poster #28, 25-27/05/2011, Noordwijk-aan-Zee (NL).
- <sup>ix</sup> VILLERS (A.), PUECH (P.), MOUTON (D.), LEROY (X.), BALLEREAU (C.), LEMAÎTRE (L.). 2006. Dynamic contrast-enhanced, pelvic phased array magnetic resonance imaging of localized prostate cancer for predicting tumor volume: Correlation with radical prostatectomy findings. Published in the Journal of Urology (JUROL). Vol. 176, Issue 6 Pt1 (December), PP.2432-2437.
- <sup>x</sup> Definitions of clinical performance indicators; applicable to the entire prostate gland or to parts of the gland according to the research question.
- Sensitivity: Test's ability to detect a cancer in a prostate gland (or in a part of the prostate gland) in which a cancer is truly present;
  - Specificity: Test's ability to conclude that there is no cancer in a prostate gland (or in a part of the prostate gland) truly free of cancer;
  - Positive predictive value (PPV): proportion of prostate glands (or in a part of prostate glands) with cancer among all prostate glands for which the test is positive;
  - Negative predictive value (NPV): proportion of prostate glands (or in a part of prostate glands) without cancer for which the test is negative.
- <sup>xi</sup> CHABANOVA (E.), BALSLEV (I.), LOGAGER (V.), HANSEN (A.), JAKOBSEN (H.), KROMANN-ANDERSEN (B.), NØRGAARD (N.), HORN (T.), THOMSEN (H.S.). 2010. Prostate cancer: 1.5T endo-coil dynamic contrast-enhanced MRI and MR spectroscopy-correlation with prostate biopsy and prostatectomy histopathological data. Published in European Journal of Radiology. Vol. 80, Issue 2 (November), PP. 292-296. doi:10.1016/j.ejrad.2010.07.004.
- <sup>xii</sup> DELONGCHAMPS (N.B.), ROUANNE (M.), FLAM (T.), BEUVON (F.), LIBERATORE (M.), ZERIB (M.), CORNUD (F.). 2011. Multiparametric magnetic resonance imaging for the detection and localization of prostate cancer: combination of T2-weighted, dynamic contrast-enhanced and diffusion weighted imaging. Published in British Journal of Urology International (BJUI). Vol. 107, Issue 9 (May), PP. 1411-1418, doi:10.1111/j.1464-410X.2010.09808.x.
- <sup>xiii</sup> NØRGAARD (N.), AUTIER (P.). 2010. Can HistoScanning™ help in the assessment of patients with raised serum PSA level: a pilot study. Published and presented at the annual meeting of the European Association of Urology (EAU). P.147, 16-20/04/2010, Barcelona (E).
- <sup>xiv</sup> ZÁT'URA (F.), KLÉZL (P.), BÁRTA (J.), AUTIER (P.). 2010. Prostate HistoScanning™ examination in patients with past negative biopsy sessions: a pilot study. Published and presented at the annual meeting of the British Association of Urological Surgeons (BAUS). P.007, 21-24/06/2010, Manchester (GB).
- <sup>xv</sup> de CONINCK (V.). 2011. The value of Prostate HistoScanning in men at risk of prostate cancer. Published thesis at the Dept. of Medicine and Pharmacy to the Vrije Universiteit Brussel (VUB), Jette (B). Promotor Dr. J. Braeckman. 24/06/2011, Jette (B). PP.1-48.
- de CONINCK (V.), BRAECKMAN (J.G.). 2012. The value of Prostate HistoScanning™. Presented as candidature at the bi-annual joint ELAUT-prijs/prix-ELAUT meeting of the Belgische Vereniging voor Urologie (BVU) and Société Belge d'Urologie (SBU). 21/01/2012, Neerpelt (B).
- <sup>xvi</sup> HAMANN (C.), SCHENK (E.J.), HAMANN (M.F.), NAUMANN (C.M.), JÜNEMANN (K.-P.). HistoScanning-guided prostate biopsy in comparison to a systemic 14-fold transrectal prostate biopsy. Published at the annual meeting of the Société internationale d'urologie (SIU). Session "Unmoderated poster session 2: BPO/LUTS, Minimally invasive surgery, Prostate cancer", P.2357.00, UP-02.116, 18/10/2011, Berlin (D).
- <sup>xvii</sup> SPETHMANN (J.), LINSE (C.), HÄSE (A.), STEUBER (T.), SCHLOMM (T.), HEINZER (H.), GRÄFEN (M.), SALOMON (G.). 2010. Accuracy of computer-aided transrectal ultrasonography detection (HistoScanning™) of prostate cancer in the prediction of a negative margin in radical prostatectomy patients. Published and presented at the annual meeting of the European Association of Urology (EAU). P.103, 16-20/04/2010, Barcelona (E).
- <sup>xviii</sup> LEMINSKI (A.), SIMMONS (L.A.M.), RASHID (T.), HAZELL (S.), MONZON (L.), WINKLER (M.). 2011. Pre-operative staging of prostate cancer with HistoScanning™ facilitates nerve sparing prostatectomy and may increase complete excision of prostate cancers. Published at the International Symposium on Focal Therapy and Imaging in Prostate & Kidney Cancer (FTI-PKc). Poster #29, 25-27/05/2011, Noordwijk-aan-Zee (NL).
- <sup>xix</sup> van den HEUVEL (S.), SIMMONS (L.A.M.), AUTIER (P.), VERHAGEN (P.), MOORE (C.), EMBERTON (M.), BANGMA (C.). 2011. Inter-observer variability in the interpretation of HistoScanning™ characterisation of prostate cancer. Published and presented at the annual meeting of the Société Internationale d'Urologie (SIU). Session "Podium Session 7: Prostate Cancer, Localized", P.1260.00, POD-07.07, Berlin (D).
- <sup>xx</sup> SIMMONS (L.A.M.), AUTIER (P.), MOORE (C.M.), EMBERTON (M.). 2011. Ultrasound spectral interrogation of histological grade in prostate cancer using Prostate HistoScanning™. Published and presented at the annual meeting of the Association of Urology (EAU). Poster Session #80 "Prostate cancer: Prognostics and treatment", P.967, 21/03/2011, Wien (A).